

UPHOLSTERY CLEANING

The Sta-Soft naugahyde is made to handle high use, with little wear or tear. Fabric cleaners used, and residual body oils table are what will break down the material faster. Therefore, please use the following guidelines when cleaning your table. Remember, a table kept dry of body oils and dampness, and only cleaned/disinfected with the following instructions will remain soft and supple for many years!

- 1) 1) ***Light soiling:*** a mild household liquid dish soap (such as Ivory dish soap) with warm water applied with a soft damp cloth. Wipe away the residue with a water dampened cloth.

- 2) ***For more difficult stains:*** Dampen soft white cloth with a solution of 10% bleach, 90% water. Rub gently. Rinse with a water dampened cloth to remove bleach concentration. Or you can allow a 1:10 diluted bleach solution to puddle on the affected area, or apply with a soaked cloth for approximately 30 minutes. Rinse with a water dampened cloth to remove any remaining bleach concentration.

It is under the advice of the manufacturer that these are the only disinfectants and cleaners to use, and they also want the cleaners to be used in conjunction with a soft white cloth, NEVER paper towels!